



Sending the CT Scan to Implant Solutions®


After scanning is complete, choose one of the options below, to send your files.

Option 1:

SEND FILES THROUGH **HIGHTAIL**


1

Go to www.solutionsforimplants.com. Click on the "upload files to Hightail" link.

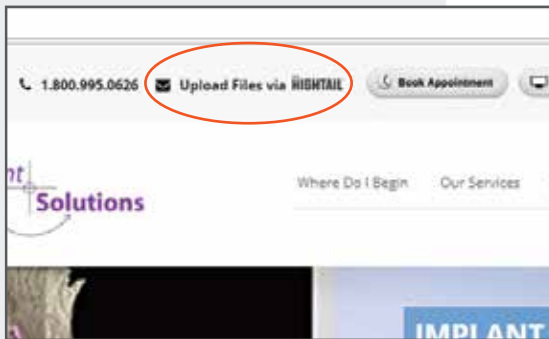


2

The Hightail main Screen will appear. Click on "My Computer" or drag your files.

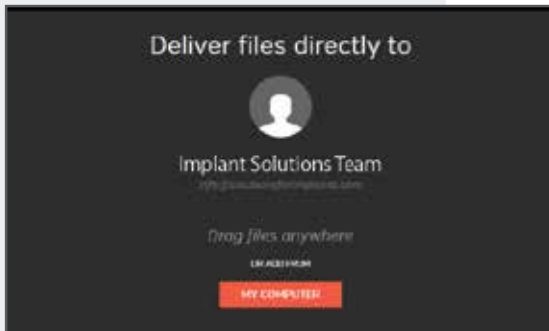


3

Select the zipped files from your computer.

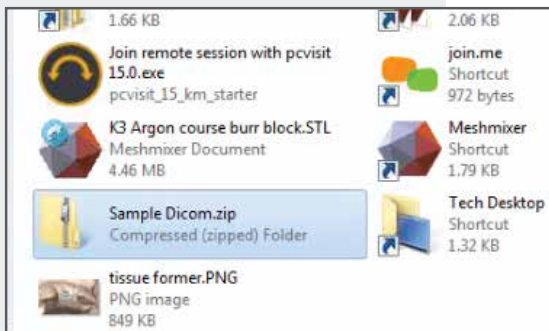


4

Insert your information: doctor's name, email, address, & message.

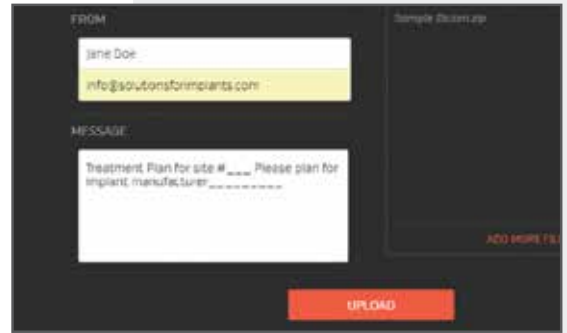


5

Please wait as your files upload to Implant Solutions®

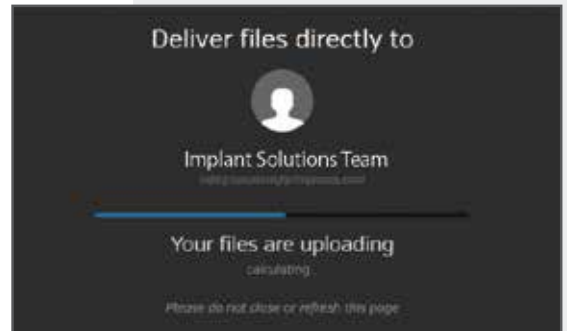

6

Upload complete.

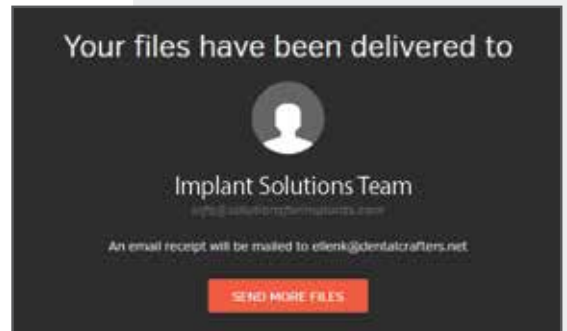

7


Doctor will receive a confirmation email once the files are received.



Option 2:

EXPORT THE DICOM FILE TO A CD OR USB FLASH DRIVE

1. Select DICOM III Multi-file Format **uncompressed**
2. Verify that the files were saved to the USB or CD
3. Send CD to Implant Solutions® at:
1000 Corporate Drive, Marshfield, WI 54449

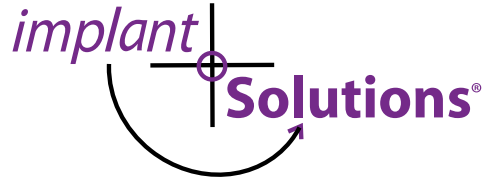

The leader in the guided implant industry with over 30,000 sites processed

Recognizing a need for affordable guided implant services, Implant Solutions, our sister company, was co-founded in 2006. Since then, we have become a leader in the guided implant industry, offering products and services that benefit doctors and their patients.

Shipping & Receiving

Please call our office for free shipping labels, boxes, Rx forms or to schedule a pickup.

Payment Options

+ CREDIT CARDS

We accept Visa, Mastercard and Discover cards.

+ CHECKS

Checks can be mailed to
Implant Solutions®
PO Box 770,
Marshfield, WI 54449.

Contact Information

IMPLANT SOLUTIONS®

1000 CORPORATE DRIVE, PO BOX 770
MARSHFIELD, WI 54449

Hours of Operation:

8:00 a.m. to 5:00 p.m. (CST)

Ph: 715.486.0626 or 800.995.0626

Fax: 715.387.4100

Email: info@solutionsforimplants.com

Website: www.solutionsforimplants.com

Questions? Please contact:

Joel A. Gonzales

Director of Sales & Service West Argon Dental USA
(480) 440-3927 | joelg@argondentalusa.com